


endo★star **PASTE FILLERS WITH SAFETY SPRING (PFL)****1. Description:**

Endostar Paste Fillers with safety spring (also called lentulo spirals, a spiral filler or a paste carrier) are used to introduce paste – like materials, cements, sealers into the root canals. The working part is conically shape and its consist of loosely coiled spiral made of thin wire. Paste Fillers with safety spring have an additional, tight spring at the metal handle, for better flexibility and protection from the breaking during the rotational movements in the canal.

- golden colour of handles allows easily to distinguish lentulo spirals from other instruments,
- application of materials is performed by clockwise movement,
- rotation speed - under 800rpm,
- cross-section: 

2. Recommended number of use:

Maximum of 1-5 times, depending on the size (see Table 1), provided that visual inspection performed by the practitioner prior to use shows that the instrument remains undamaged, is not bent (does not apply to bending the instrument by dentist to the curvature of the canal), deformed, does not show signs of blade wear. Prolonging the life of the instrument more than recommended may result in the blade breaking.

Table 1

File No	25	30	35	40
Number of times that the instrument can be used:	1-2	4-5	4-5	4-5

3. Cleaning and disinfection:

Detailed instructions for cleaning and disinfection can be found at www.poldent.pl and www.endostar.eu.

4. Sterilization:









This is a non-sterile product. Sterilize before use. The instruments can be sterilized in a steam sterilizer (autoclave) at 134°C. Recommended sterilization time 35-40 minutes. Instruments can be disinfected with mild disinfectants and washed in ultrasonic cleaners. Very aggressive disinfectants may cause a marking discoloration of the shank.

5. Storage:

Instruments should be stored at room temperature in a dry, dust-free and clean environment.

6. Warnings:

Only for dental use.

	Cross-section
	CE mark and identification number of the notified body
	Sterilize in a steam autoclave at 134 ° C
	Non-sterile product
	Used for root canal preparation.
	Serial number/ Lot number
	Consult instruction for use
	Stainless steel



Manufactured by:

Poldent Co. Ltd.

Dzika 2 Street

00-194 Warsaw, POLAND

tel.: +48 22 351-76-50 do 51

fax: +48 22 351-76-79

www.poldent.pl, www.endostar.eu, e-mail: poldent@poldent.pl, endostar@endostar.eu



This manual was last updated: 01.2018