



CleaniCal®

Hydrosoluble Calcium Hydroxide Paste

2 g Syringe X 1 EA + 24 gauge Tip X 20 EA + Storage cap X 5 EA

PRODUCT DESCRIPTION

CleaniCal is a preloaded, creamy calcium hydroxide paste in syringe to quickly and easily dispense into the root canal.

PURPOSE OF USE

Intracanal medicament for root canal

PHYSICAL, CHEMICAL AND BIOLOGICAL PROPERTIES (according to ISO 6876)

- Flow : above 17 mm
- Radiopacity : The same radiopacity as that of aluminum of above 3 mm in thickness

PRECAUTIONS FOR USE

1. WARNINGS

- 1) Do not use in patients with allergic reaction to chemicals.
- 2) For dentists use only.
- 3) Do not use the product as a medicine.


2. GENERAL PRECAUTIONS

- 1) Always check if the package is damaged before use.
- 2) Do not use for other purposes.
- 3) Contact with the skin and eyes should be avoided, and in case of contact, immediately rinse with running water.
- 4) Avoid exposure to direct sunlight.
- 5) Store at room temperature.

3. APPLICATIONAL PRECAUTIONS

- 1) Inject to middle third of the root canal and do not overdose beyond it.
- 2) Apply to the apical third of the root canal with using a gutta percha cone.
- 3) In case of treatment, use the product after replacement with a new 24 gauge tip.
- 4) Do not carry out heating sterilization before and after use.

CE
0197

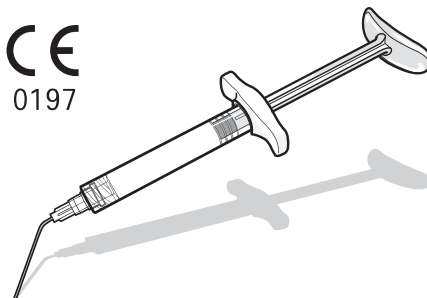

DIRECTION FOR USE

1. PREPARATION PRIOR TO USE

- 1) Always check if the package is damaged before use.
- 2) Protect the patient mouth with the use of a rubber dam.
- 3) Confirm the expiration date.

2. DIRECTION FOR USE


- ① Dry the root canal to be filled.
- ② Replace with a new 24 gauge tip.
- ③ Check if the paste releases smoothly from the 24 gauge tip.
- ④ Insert the 24 gauge tip at the middle third of the root canal.
- ⑤ Inject the paste into the root canal
- ⑥ Using GP cones, push the paste to the working length.
- ⑦ Take radiographs.

STORAGE

- 1) The product must be stored at room temperature avoiding high temperature and humidity.
- 2) A new 24 gauge tip must be used in next use.
- 3) After use do not recap the used 24 gauge tip, twist the storage cap provided with the product tightly onto the syringe head.
- 4) Once the gray cap, which was initially fastened to the syringe, is separated, it will not be sealed even if it is fastened again. It must be sealed using the enclosed storage cap.
- 5) Place syringe with the storage cap in the provided pouch. Seal the pouch completely.
- 6) Tighten the storage cap immediately after use to minimize the product from contamination and infection during storage.

SHELF LIFE

See expiration date on the package.

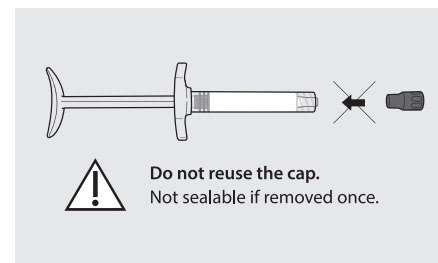

SYMBOLS

- | | |
|--|--|
| | Non-sterile |
| | Temperature limit |
| | Consult instructions for use |
| | Batch code |
| | Date of manufacture |
| | Use-by date |
| | Manufacturer |
| | Authorized representative in the European community |
| | Caution |
| | Do not use if package is damaged |
| | Keep away from sunlight |
| | Keep dry |

We just mimic nature
MARUCHI
www.endoland.com

MARUCHI
101, Donghwagongdan-ro, Munmak-eup, Wonju-si,
Gangwon-do, 26365 Republic of Korea
TEL : +82-33-734-0330
FAX : +82-33-746-2804

KTR Europe GmbH
Mergenthalerallee 77, 65760, Eschborn, Germany
TEL: +49-6196-887-170
Fax: +49-6196-887-1728